



1 box with 10 applicators can treat up to 30 lesions

CURODONT™ REPAIR FLUORIDE PLUS


Biomimetic system for Guided Enamel Remineralization of early caries

A unique brush-on, biomimetic system that tackles early caries through Guided Enamel Remineralization.

- Works on all surfaces
- Treats initial caries with no injection and drilling
- Helps improve longevity by naturally preserving the tooth
- Non-staining, pain-free and tasteless
- One-time application
- Suitable for all patients, including children
- Easy to use in as little as 5 minutes, by dentists, hygienists and support staff*

*state-dependent

Dental Practice with 3 Registered Dental Hygienists

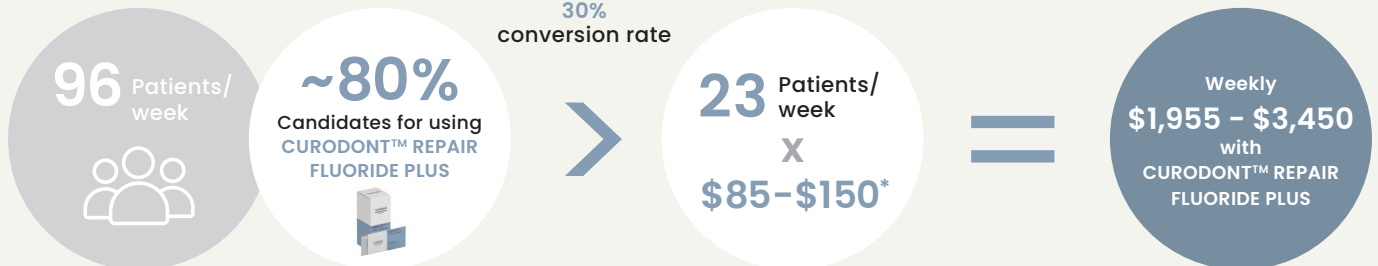

*average charge per application (treating up to 3 lesions); average of what is being charged in dental practices

GUIDED ENAMEL REMINERALIZATION – NON-INVASIVE, EASY AND PAIN-FREE

Guided Enamel Remineralization closes the gap between simple prevention and invasive restorations. In tackling white spots, dental professionals can offer their patients an anticavity therapy that is non-invasive, easy and pain-free. CURODONT™ REPAIR FLUORIDE PLUS is an effective treatment that helps preserve the natural teeth especially for dental professionals' younger patients.

MODE OF ACTION

CURODONT™ REPAIR FLUORIDE PLUS brings fluoride where it is needed most – directly to the white spots. The technology's formulation helps minerals, like phosphate and calcium found in the saliva, to diffuse the enamel surface into the depth of the white spot. Lost enamel is re-hardened and remineralized over the next few weeks.



Scan the QR code to access the CURODONT™ REPAIR FLUORIDE PLUS application videos and other educational materials



Each box contains ten 3ml individually-packaged tubes

CURODONT™ PROTECT Anti-cavity & Anti-sensitivity Dental Gel

A remineralizing dental gel that helps with cavity prevention and builds increasing protection against tooth sensitivity


- Aids in the prevention of dental cavities
- Remineralizes and strengthens enamel
- Ideal at home complement to CURODONT™ REPAIR FLUORIDE PLUS
- Builds increasing protection against painful sensitivity of the teeth
- Suitable for all patients over 6 years of age
- Easy application in office and at home

INDICATIONS

- **Cavity prevention**, particularly for orthodontic patients and for those with a high risk of caries
- **After the treatment with CURODONT™ REPAIR FLUORIDE PLUS** in the dental office and at home
- **Sensitivity relief** before and after professional hygiene treatments/ bleaching sessions/ for periodontal patients

MODE OF ACTION

The concentrated formula is designed to:

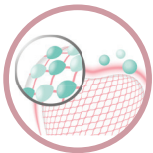

Support the uptake of calcium and phosphate from saliva on the tooth surface for remineralization



Release stabilized stannous fluoride to protect the sensitive areas of the tooth against cold, heat, acids, sweets or contact

APPLICATION IN OFFICE

- **Before professional cleaning /bleaching etc.:** Apply 2 minutes before the dental procedure for sensitivity relief.
- **After treatment with CURODONT™ REPAIR FLUORIDE PLUS:** Wait for 5 minutes and apply on treated areas and on the rest of the teeth.

APPLICATION AT HOME

- After treatment with CURODONT™ REPAIR FLUORIDE PLUS: Once a day, after brushing at bedtime, for 1 week or until the tube is finished.
- For caries prevention: Twice a week, after brushing at bedtime.
- For sensitivity relief: One or more times a day until the pain subsides.

MAXIMIZE OUTCOMES WITH THE V VARDIS CURODONT™ SYSTEM FOR WATCH AREAS MANAGEMENT

Treat with
CURODONT™ REPAIR FLUORIDE PLUS
in-office



Maintain & Protect with
CURODONT™ PROTECT
in-office and/or at-home



To learn more about CURODONT™ professional products, simply scan the QR code for our team's contact details