

vVARDIS

SWITZERLAND

BIOMIMETIC DENTAL SCIENCE



CURODONT™ PROTECT Anti-cavity & Anti-sensitivity Dental Gel

HOW TO APPLY CURODONT™ PROTECT IN-OFFICE

CURODONT™ PROTECT is quick and easy to apply with the help of a rubber polisher, interdental brush, or a micro-brush.

Before professional cleaning/bleaching or any invasive treatment

Apply 2 minutes before the dental procedure to reduce sensitivity and discomfort during the visit.

After treatment with
CURODONT™ REPAIR FLUORIDE PLUS

Wait for 5 minutes and apply on treated areas and on the rest of the teeth.

HOW TO APPLY CURODONT™ PROTECT AT HOME

CURODONT™ PROTECT can be applied with a toothbrush, interdental brush, or a clean finger.

After treatment with
CURODONT™ REPAIR FLUORIDE PLUS

Once a day, after brushing at bedtime, for 1 week or until the tube is finished.

For caries prevention

Twice a week, after brushing at bedtime.

For sensitivity relief

One or more times a day until the pain subsides.

Please note: The gel must be allowed to stay on the teeth undisturbed for 1-2 minutes before spitting out the residue, if any. This should not be followed by rinsing to maintain the contact of the gel with the teeth for as long as possible.

MAXIMIZE OUTCOMES WITH THE vVARDIS CURODONT™ SYSTEM FOR WATCH AREAS MANAGEMENT

Treat with
CURODONT™ REPAIR FLUORIDE PLUS
in-office



Maintain & Protect with
CURODONT™ PROTECT
in-office and/or at-home